

Your Smile

Is a Big Part of Your Day.

Why let wrinkles get in the way?

JUVÉDERM® XC can instantly smooth away moderate to severe facial wrinkles and folds.

Your doctor can

Style Your Smile.[™]

Ask how today!

JUVÉDERM® XC Important Information

Indication

JUVÉDERM® injectable gel is injected into areas of facial tissue where moderate to severe facial wrinkles and folds occur to temporarily add volume to the skin, which may give the appearance of a smoother surface.

IMPORTANT SAFETY INFORMATION

Your physician will ask about your medical history to determine if you are an appropriate candidate for treatment. The product should not be used in patients who have:

- Severe allergies marked by a history of anaphylaxis or history or presence of multiple severe allergies
- A history of allergies to lidocaine or Gram-positive bacterial proteins

Please see additional important safety information on following pages.

Your Smile Is Too Important to Ignore

Four types of facial wrinkles and folds—**parentheses lines**, **vertical lip lines**, **corner lines**, and **marionette lines**—can form around your nose and mouth. They may appear deeper when you smile. And when you are not smiling, they can even make people think you are sad.

JUVÉDERM® XC can smooth the wrinkles and folds around your smile

You may have heard about how well JUVÉDERM® XC corrects **parentheses lines**. The US and world's #1 selling dermal filler^{1,2} can also smooth moderate to severe facial wrinkles and folds such as **vertical lip lines**, **corner lines**, and **marionette lines** that appear around your nose and mouth.

Ask your doctor if JUVÉDERM® XC is right for you.

IMPORTANT SAFETY INFORMATION (continued)

The safety and effectiveness for the treatment of areas other than facial wrinkles and folds (such as lips) have not been established in controlled clinical studies.

The following are important treatment considerations for you to discuss with your physician and understand in order to help avoid unsatisfactory results and complications:

Please see additional important safety information on following pages.

Parentheses lines*



Before

After 2 weeks
JUVÉDERM® Ultra (0.5 mL)
JUVÉDERM® Ultra Plus (1.5 mL)

Vertical lip lines*



Before

After 2 weeks
JUVÉDERM® Ultra (0.1 mL)
JUVÉDERM® Ultra Plus (0.2 mL)


Corner lines*



Before

After 2 weeks
JUVÉDERM® Ultra (0.2 mL)
JUVÉDERM® Ultra Plus (0.2 mL)

Marionette lines*

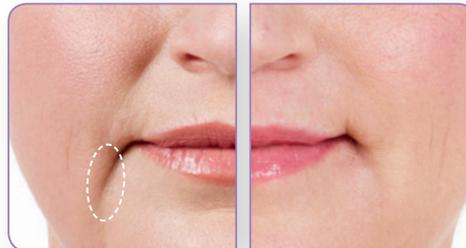

Before

After 2 weeks
JUVÉDERM® Ultra (1.05 mL)
JUVÉDERM® Ultra Plus (1.35 mL)

Individual results may vary. Models were paid for their appearance. Photos taken 2 weeks after treatment. Models may have been treated in more than one area. These photos are not of clinical trial subjects.

*JUVÉDERM® injectable gel is injected into areas of facial tissue where moderate to severe facial wrinkles and folds occur to temporarily add volume to the skin, which may give the appearance of a smoother surface.

JUVÉDERM® XC Provides Instant, Personal Results

You don't have to worry about a one-size-fits-all treatment

That's because your doctor can choose the formulation that can instantly meet your needs:

- **JUVÉDERM® Ultra XC** for versatility in filling moderate to severe facial wrinkles and folds
- **JUVÉDERM® Ultra Plus XC**, a more robust formulation, for volumizing and correcting deeper facial wrinkles and folds

JUVÉDERM® injectable gel is injected into areas of facial tissue where moderate to severe facial wrinkles and folds occur to temporarily add volume to the skin, which may give the appearance of a smoother surface.

**Talk to your doctor about
a look that's right for you.**

IMPORTANT SAFETY INFORMATION (continued)

- Patients who are using substances that can prolong bleeding, such as aspirin or ibuprofen, as with any injection, may experience increased bruising or bleeding at injection site. You should inform your physician before treatment if you are using these types of substances

Please see additional important safety information on following pages.



Before

After 2 weeks

Model was treated for parentheses lines (JUVÉDERM® Ultra, 0.5 mL; JUVÉDERM® Ultra Plus, 1.5 mL), marionette lines (JUVÉDERM® Ultra, 0.3 mL; JUVÉDERM® Ultra Plus, 0.5 mL), and corner lines (JUVÉDERM® Ultra, 0.2 mL; JUVÉDERM® Ultra Plus, 0.2 mL).



Before

After 2 weeks

Model was treated for vertical lip lines (JUVÉDERM® Ultra, 0.1 mL; JUVÉDERM® Ultra Plus, 0.2 mL), parentheses lines (JUVÉDERM® Ultra, 0.5 mL; JUVÉDERM® Ultra Plus, 0.95 mL), corner lines (JUVÉDERM® Ultra, 0.15 mL; JUVÉDERM® Ultra Plus, 0.15 mL), and marionette lines (JUVÉDERM® Ultra, 0.25 mL; JUVÉDERM® Ultra Plus, 0.7 mL).

Individual results may vary. Models were paid for their appearance. Photos taken 2 weeks after treatment. Models may have been treated in more than one area. These photos are not of clinical trial subjects.

JUVÉDERM® XC: Smooth and Comfortable

The JUVÉDERM® XC difference

- **SMOOTH**—Only JUVÉDERM® XC is manufactured using HYLACROSS™ technology, creating a smooth-consistency gel that flows easily into the skin³
- **COMFORTABLE**—And because JUVÉDERM® XC is infused with lidocaine, a common anesthetic, you'll be more comfortable during treatment (compared to the nonlidocaine JUVÉDERM® formulations)⁴

Ask your doctor about the
JUVÉDERM® XC difference.



Before




After 2 weeks

Model was treated for corner lines (JUVÉDERM® Ultra, 0.2 mL; JUVÉDERM® Ultra Plus, 0.2 mL), parentheses lines (JUVÉDERM® Ultra, 0.5 mL; JUVÉDERM® Ultra Plus, 0.5 mL), and marionette lines (JUVÉDERM® Ultra, 0.3 mL; JUVÉDERM® Ultra Plus, 0.3 mL).



Before

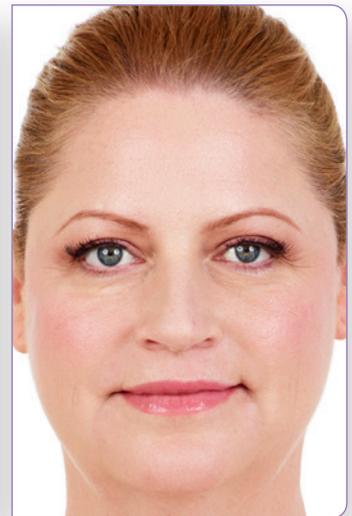

After 2 weeks

Model was treated for marionette lines (JUVÉDERM® Ultra, 1.05 mL; JUVÉDERM® Ultra Plus, 1.35 mL), parentheses lines (JUVÉDERM® Ultra, 0.8 mL; JUVÉDERM® Ultra Plus, 0.3 mL), corner lines (JUVÉDERM® Ultra, 0.25 mL; JUVÉDERM® Ultra Plus, 0.25 mL), and vertical lip lines (JUVÉDERM® Ultra, 0.1 mL; JUVÉDERM® Ultra Plus, 0.1 mL).

IMPORTANT SAFETY INFORMATION (continued)

- If laser treatment, chemical peeling, or any other procedure based on active dermal response is considered after treatment with JUVÉDERM®, there is a possible risk of an inflammatory reaction at the treatment site
- JUVÉDERM® injectable gel should be used with caution in patients on immunosuppressive therapy, or therapy used to decrease the body's immune response, as there may be an increased risk of infection

Please see additional important safety information on following pages.

Individual results may vary. Models were paid for their appearance. Photos taken 2 weeks after treatment. Models may have been treated in more than one area. These photos are not of clinical trial subjects.

The Look That Lasts All Year—With Initial Treatment⁵

And we know that's important to you

JUVÉDERM® XC is the *first* and *only* FDA-approved HA filler that offers smooth and natural-looking results that last up to 1 year with 1 treatment when you treat moderate to severe facial wrinkles and folds such as parentheses lines, vertical lip lines, corner lines, and marionette lines that appear around your nose and mouth.⁵ Most patients achieved desired correction at initial treatment.

Your doctor can
Style Your Smile™
today—ask how!

IMPORTANT SAFETY INFORMATION (continued)

- The safety for use during pregnancy, in breast-feeding females, or in patients under 18 years has not been established
- The safety in patients with a history of excessive scarring (eg, hypertrophic scarring and keloid formations) and pigmentation disorders has not been studied

Please see additional important safety information on following pages.

Visualize the Difference With



The app is available only for Apple products (iPad®, iPhone®, and iPod touch®).

The JUVÉDERM® XC Treatment Visualizer app is for illustrative and demonstrative purposes only. The results displayed may not reflect actual results.

Download the Treatment Visualizer app

- Upload your photo to see your potential JUVÉDERM® XC results
- Shows possible JUVÉDERM® XC treatment areas
- Choose from 3 levels of correction
- Schedule an appointment with your doctor to see if JUVÉDERM® XC is right for you

Scan here to get the Treatment Visualizer app now.



IMPORTANT SAFETY INFORMATION (continued)

Most side effects are mild or moderate in nature, and their duration is short lasting (7 days or less). The most common side effects include, but are not limited to, temporary injection-site reactions such as: redness, pain/tenderness, firmness, swelling, lumps/bumps, bruising, itching, and discoloration.

As with all skin-injection procedures, there is a risk of infection.

Please see additional important safety information on following pages.

Join our rewards program
and start earning today!

Brilliant
DISTINCTIONS®
PROGRAM

- **Earn points** each time you get JUVÉDERM® XC treatments
- **Redeem your points** for big savings on JUVÉDERM® XC and other Allergan products

Reward yourself. You've earned it.

IMPORTANT SAFETY INFORMATION (continued)

To report a problem with JUVÉDERM®, please call Allergan Product Surveillance at 1-800-624-4261.

For more information, please see the About Safety page at www.juvederm.com or call the Allergan Product Support line at 1-800-433-8871.

JUVÉDERM® injectable gel is available by prescription only.

Ask your doctor today!



Bibliography:

1. Data on file, Allergan, Inc., July 23, 2012; US Facial Injectables Market Share Report, Q2 2012. 2. Data on file, Allergan, Inc.; Worldwide Dermal Facial Filler Market Overview, Q2 2011. 3. Tezel A, Fredrickson GH. The science of hyaluronic acid dermal fillers. *J Cosmet Laser Ther.* 2008;10(1):35-42. 4. Weinkle SH, Bank DE, Boyd CM, Gold MH, Thomas JA, Murphy DK. A multi-center, double-blind, randomized controlled study of the safety and effectiveness of JUVÉDERM® injectable gel with and without lidocaine. *J Cosmet Dermatol.* 2009;8(3):205-210. 5. JUVÉDERM® Ultra Plus XC Directions for Use.



©2013 Allergan, Inc. ® and ™ marks owned by Allergan, Inc.
JUVÉDERM® mark owned by Allergan Industrie, SAS. iPad, iPhone, and iPod touch are trademarks of Apple Inc., registered in the US and other countries. Radiesse is a registered trademark of Merz Aesthetics, Inc.
www.juvederm.com Re-order: APC05KX13 123340